Prior to use, carefully read the instructions for use.

EN

GC Gold Label HYBRID

GLASS HYBRID SELF-ADHESIVE POSTERIOR RESTORATIVE

For use only by a dental professional in the recommended indications.

INDICATIONS FOR USE

- Class I and II restorations in deciduous teeth.
- Non-load bearing Class I and Class II.
- Intermediate restorative and base material for heavy stress situation in Class I and Class II cavities using sandwich laminate technique.
- Class V and root surface restorations.
- Core build-up.

CONTRAINDICATIONS

- Direct pulp capping.
- Avoid use of this product in patients with known allergies to glass ionomer cement.

DIRECTION FOR USE

Powder / Liquid Ratio (g/g)	3.0 / 1.0
Mixing Time (sec.)	25-30"
Working Time (23°C/73°F) (min., sec.)	1'45"
Net Setting Time (37°C/99°F) (min., sec.)	2'40"
Finishing Time (37°C/99°F, from start of mixing) (min., sec.)	6'00"

1. POWDER AND LIQUID DISPENSING

- The standard powder to liquid ratio is 3.0 g / 1.0 g (1 level scoop of powder to 1 drop of liquid).
- For accurate dispensing of powder, lightly tap the bottle against the hand. Do not shake or invert.
- Hold the liquid bottle vertically and squeeze gently.
- Immediately close the bottles tightly after use.

2. CAVITY PREPARATION

- Prepare tooth using standard techniques. Extensive mechanical retention is unnecessary. For pulp capping, use calcium hydroxide.

Note:

Optionally, CAVITY CONDITIONER or DENTIN CONDITIONER can be applied to the bonding surfaces.

- Rinse thoroughly with water. Blot away excess water with a cotton pellet or dry by gently blowing with an air syringe. DO NOT DESICCATE. Prepared surfaces should appear moist (glistening) (Fig. 1).

3. MIXING







- Dispense powder and liquid onto the pad.
- Using plastic spatula, divide the powder into 2 equal parts.
- Mix the first portion of the powder with all the liquid for 10 seconds.
- Incorporate the remaining portion of the powder and mix thoroughly for 15-20 seconds (Fig. 2).

4. RESTORATIVE TECHNIQUE

- Mix the required amount of cement. Working time is 1'45" from start of mixing at 23°C (73°F). Higher temperatures will shorten working time.
- Transfer cement to the prepared cavity using a suitable instrument. Avoid air bubbles.
- Form the preliminary contour and cover with a matrix if required (Fig. 3).
- When set, immediately apply GC Fuji VARNISH (blow dry) or G-COAT PLUS (light cure) (Fig. 4).

5. FINISHING

- Final finishing under water spray using standard techniques can begin at 6 minutes after starting the mix (Fig. 5).
- Apply a final coat of GC Fuji VARNISH or G-COAT PLUS (Fig. 6).
- Instruct the patient not to apply pressure for 1 hour.

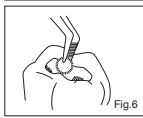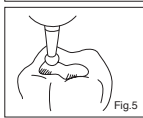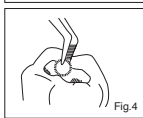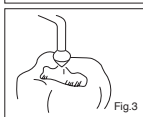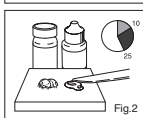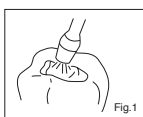STORAGE

Recommended for optimal performance, store at temperature of 4-25°C (39-77°F).

SHADES

A2, A3, A3.5

Shade numbers according to Vita® shade guide

Vita® is a registered trademark of Vita Zahnfabrik, Bad Säckingen, Germany.

PACKAGES

1-1 Package: 15 g powder, 8 g (6.4 mL) liquid, powder scoop, mixing pad (No. 22)

CAUTION

- In rare cases, the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- DO NOT mix powder or liquid with any other glass ionomer components.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
- DO NOT reuse the applicators for Conditioner to other patients.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

<http://www.gcamerica.com>

<http://sea.gcasiadental.com>

They can also be obtained from your supplier.

CLEANING AND DISINFECTING:

MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.

DO NOT IMMERSE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

Last revised: 10/2021

GC

MANUFACTURED BY
GC CORPORATION
76-1 Hasunuma-cho, Habashi-ku, Tokyo 174-8585, Japan

DISTRIBUTED BY
GC AMERICA INC.
3737 West 127th Street, Aleip, IL 60803 U.S.A.
TEL: +1-708-597-0900
www.gcamerica.com

GC SOUTH AMERICA
Rua Heliadora, 399, Santana - São Paulo, SP, BRASIL
CEP: 02022-051 - TEL: +55-11-2925-0965
CNPJ: 08.279.999/0001-61
RESP. TEC: Erick de Lima - CRO/SP 100.866

GC ASIA DENTAL PTE. LTD.
5 Tampines Central 1, #06-01 Tampines Plaza 2, Singapore 529541
TEL: +65 6546 7588

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